



## ProScreen Cup Instructions



✓ **STEP 1**

Collect Specimen in cup – collector retains the lid



✓ **STEP 2**

Verify temperature is within range (32-38 C)



✓ **STEP 3**

Place the lid on the cap and make sure it's tightly sealed




✓ **STEP 4**

Place cup on its side, so it's resting on the "feet" in the cap.

Read results when the lines

# ProScreen Results Interpretation

## Reading the Result Windows

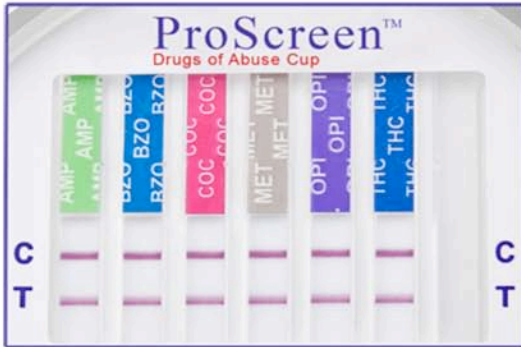

**NEGATIVE**

A Negative test is indicated by the presence of a test line below each designated drug.




**NEGATIVE**

This is also a Negative screen. Even a **very light** colored line indicates a negative result.



**POSITIVE**

A Positive test is indicated by the **ABSENCE** of a Test Line below the designated drug, with the Control line present for that channel.

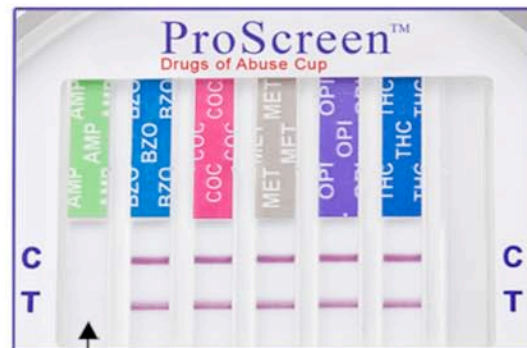

**INVALID**

An Invalid test is indicated when the "C" Control Lines are completely missing in one or more of the test windows. In event of an invalid screen a second screen should be run.

**Wait 5 minutes before reading Positive Results. Do not read results past 10 Minutes.**

**AustraliaDrugTesting.com, Pty Ltd**

**Telephone: 1800 187 238**

**www.AustraliaDrugTesting.com**