



## ProScreen Cassette Instructions



✓ **STEP 1**

**Collect Specimen**




✓ **STEP 2**

**Verify temperature is within range (32-38 C)**



✓ **STEP 3**

**Apply 3 drops of specimen to each sample well**

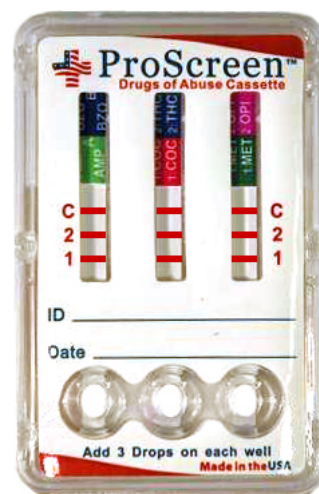

✓ **STEP 4**

**Test will run in 1-2 minutes.  
Read result when lines appear.**

# ProScreen Results Interpretation

## Reading the Result Windows



**NEGATIVE**

A Negative test is indicated by the presence of a test line below each designated drug.




**NEGATIVE**

This is also a Negative screen. Even a **very light** colored line indicates a negative result.



**POSITIVE**

A Positive test is indicated by the **ABSENCE** of a Test Line below the designated drug, with the Control line present for that channel.

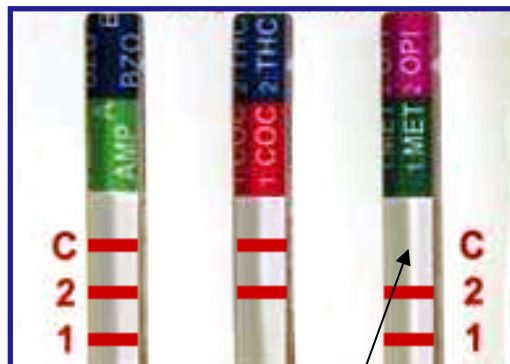

**INVALID**

An Invalid test is indicated when the "C" Control Lines are completely missing in one or more of the test windows. In event of an invalid screen a second screen should be run.

***Wait 5 minutes before reading Positive Results. Do not read results Past 10 Minutes.***

**AustraliaDrugTesting.com, Pty Ltd**

**Telephone: 1800 187 238**

**www.AustraliaDrugTesting.com**