



ProScreen Dip Instructions



✓STEP 1

Collect Specimen and verify temperature is within range (32-38 C)



✓STEP 2

Remove lower cap and dip test fingers into specimen



✓STEP 3

Once all windows begin to turn pink, remove ProScreen Dip from the specimen and replace the cap.




✓STEP 4

Interpret Adulteration Strip and Test Results


ProScreen Results Interpretation

Reading the Result Windows

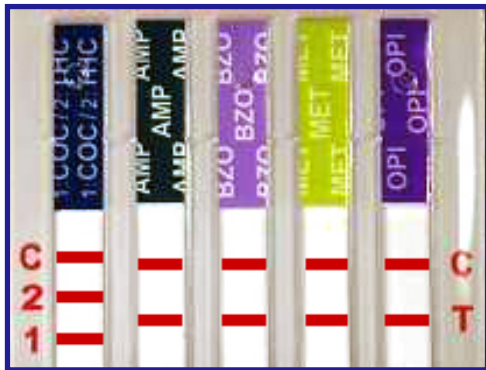


NEGATIVE

A Negative test is indicated by the presence of a test line below each designated drug.

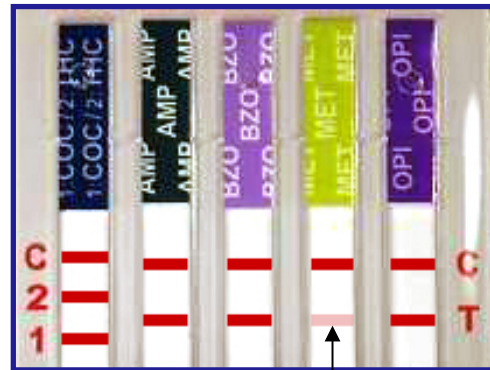


NEGATIVE

This is also a Negative screen. Even a **very light** colored line indicates a negative result.

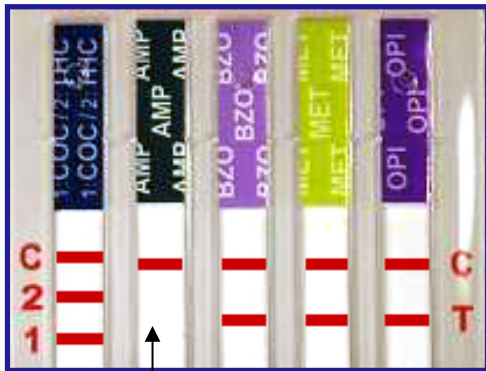

POSITIVE

A Positive test is indicated by the **ABSENCE** of a Test Line below the designated drug, with the Control line present for that channel.

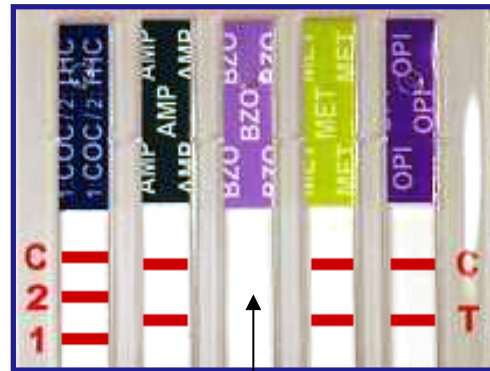

INVALID

An Invalid test is indicated when the "C" Control Lines are completely missing in one or more of the test windows. In event of an invalid screen a second screen should be run.

Wait 5 minutes before reading Positive Results. Do not read results Past 10 Minutes.

AustraliaDrugTesting.com, Pty Ltd

Telephone: 1800 187 238

www.AustraliaDrugTesting.com