



EcoCup II Instructions

Step 1: Remove EcoCup II from foil pouch.

Step 2: Remove cup lid and provide urine specimen directly into the cup.



Step 3: After urine specimen has been provided into the cup, replace and secure the cup lid.

Test will now start to run.
Result will be complete in 5 minutes.



Step 4: Peel off label to reveal the test result.



How to Interpret the Results:

- Test is valid if there is a line next to "C"
- A line next to "T" = negative no matter how faint it is
- The absence of a line next to "T" = positive



DRUG GUIDE:

THC = Marijuana
COC = Cocaine
AMP = Amphetamine
MET = Methamphetamine
OPI = Opiates
BZO = Benzodiazepines